

# THE RENUVION® BODY



REAL  
PATIENTS

renuvion®

#THISISME



REAL  
PATIENTS

REAL  
RESULTS

# REAL RESULTS

## Let's talk body contouring.

It's no surprise that liposuction is now the number one body contouring procedure in the world.<sup>1</sup> This procedure effectively removes stubborn fat, but it doesn't address the problem of the loose skin that happens when we age or lose weight. In fact, liposuction can even increase skin laxity, because of the decreased volume of fat.

## That's where Renuvion fits in.

Renuvion goes beneath the surface to address the problem directly at the source.

The wand is inserted under the skin and delivers a unique combination of radiofrequency energy and helium plasma to the collagen-rich tissue that's a primary cause of loose skin. This energy contracts these collagen fibers and pulls the skin down, closer to the muscle, for a more contoured appearance.

Renuvion is the **only** device that is FDA-cleared for use after liposuction, and the **only** device FDA-cleared for contracting subcutaneous soft tissue.<sup>2</sup>



Before

**Patient:** 40-year-old female  
**Procedure Type:** Liposuction & Renuvion  
**Amount of Fat Removed:** 40cc  
*Photos courtesy of Richard Zeff, MD*



1 Week Post-Op



Before

**Patient:** 40-year-old, mother of two  
**Procedure Type:** Liposuction & Renuvion  
**Amount of Fat Removed:** 150cc  
*Photos courtesy of Vaishali Doolabh, MD*  
*\*No change in weight, diet or exercise.*



2 Years Post-Op\*



Before

**Patient:** 33-year-old female  
**Procedure Type:** Liposuction & Renuvion  
**Amount of Fat Removed:** 1750cc  
*Photos courtesy of J. Kevin Duplechain, MD*



2½ Months Post-Op



Before

**Patient:** 22-year-old female  
**Procedure Type:** Liposuction & Renuvion  
**Amount of Fat Removed:** 1800cc  
*Photos courtesy of Emil Kohan, MD*  
*\*Patient also had fat transfer to buttocks*



6 Weeks Post-Op\*



Before

**Patient:** 42-year-old mother of two  
**Procedure Type:** Liposuction & Renuvion  
**Amount of Fat Removed:** 175cc  
*Photos courtesy of Richard Zeff, MD*



10 Weeks Post-Op



Before

**Patient:** 53-year-old female  
**Procedure Type:** Liposuction & Renuvion  
**Amount of Fat Removed:** 1100cc  
*Photos courtesy of Melinda Lacerna, MD*



5 Weeks Post-Op

# FREQUENTLY ASKED QUESTIONS

## Q: Are results long-lasting?

**A:** We can't exactly stop the aging process, but through the use of energy devices, like Renuvion, it's possible to contract soft tissues and start the long-lasting collagen rebuilding process known as neocollagenesis.<sup>2,3</sup>

## Q: How many treatments are needed?

**A:** Renuvion is a single treatment. Results may be seen in as little as days or weeks and will continue to improve throughout the healing process.

## Q: How long does the Renuvion procedure take?

**A:** Depending on the area of the body treated, the use of Renuvion only adds a few minutes per area to a standard liposuction case.

## EXPERIENCE YOU CAN TRUST

Apyx® Medical is deeply rooted in the medical industry. We bring the same scientific rigor to cosmetic medicine, using evidence-based research to establish and validate claims, and to educate clinicians and patients on what to expect from their Renuvion experience.

## DISCLAIMERS/RISKS

The Renuvion APR Handpiece is intended for the delivery of radiofrequency energy and/or helium plasma where coagulation/contraction of soft tissue is needed. Soft tissue includes subcutaneous tissue. The Renuvion APR Handpiece is intended for the coagulation of subcutaneous soft tissues following liposuction for aesthetic body contouring. The Renuvion APR Handpiece is indicated for use in subcutaneous dermatological and aesthetic procedures to improve the appearance of lax (loose) skin in the neck and submental region. The Renuvion APR Handpiece is intended for the delivery of radiofrequency energy and/or helium plasma for cutting, coagulation and ablation of soft tissue during open surgical procedures. The Renuvion APR Handpiece is intended to be used with compatible electrosurgical generators owned by Apyx® Medical. Not all indications are approved in all markets; check with your local sales representative for further information.

As with any aesthetic procedure, individual results may vary. As with all energy devices there are inherent risks associated with its use. Risk associated with the use of the Renuvion APR may include: helium embolism into the surgical site due to inadvertent introduction into the venous or arterial blood supply system, unintended burns (deep or superficial), pneumothorax, temporary or permanent nerve injury, ischemia, fibrosis, infection, pain, discomfort, gas buildup resulting in temporary and transient crepitus or pain, bleeding, hematoma, seroma, subcutaneous induration, pigmentation changes, increased healing time, scarring, asymmetry and/ or unacceptable cosmetic result. Please see the instructions for use for more detailed information.

### References:

1. ISAPS International Survey on Aesthetic/Cosmetic Procedures 2021. [https://www.isaps.org/media/vdpdanke/isaps-global-survey\\_2021.pdf](https://www.isaps.org/media/vdpdanke/isaps-global-survey_2021.pdf)
2. FDA 510(k) Premarket Notifications K230272 & K223262.
3. Duncan DI and Roman S. Helium Plasma Subdermal Tissue Contraction Method of Action. Biomed J Sci & Tech Res 31(2)-2020. BJSTR. MS.ID.005075.



Apyx Medical  
Clearwater, FL USA  
+1 800 537 2790  
[www.renuvion.com](http://www.renuvion.com)

Follow us @Renuvion



LEARN MORE

©Copyright 2023 Apyx Medical. All rights reserved.  
Apyx® and Renuvion® are registered trademarks of  
Apyx Medical Corporation.

MM0181.05 0523